



'5997954(240)US-H215 003-2022

US-H215_003-2022-05 MAR-01360, Vers. 3

Rx only

LINK MP Monoblock Hip Prosthesis Instructions for Use Implants & Instruments

For U.S. Distribution Only

For further information please refer to the U.S. section of our website:

www.ifu-us.linkorthopaedics.com





LinkBio Corp.

69 King Street | Dover NJ 07801, USA info@linkbio.com | www.linkorthopaedics.com/us Phone number: +1 973-625-1333



Waldemar Link GmbH & Co. KG

Barkhausenweg 10 22339 Hamburg Germany info@linkhh.de www.linkorthopaedics.com Tel.: +49 (0) 40 5 39 95 - 0 Fax: +49 (0) 40 5 38 69 29

Legend of label symbols and descriptions

	Manufacturer
US DIST	U.S. Distributor
\triangle	Caution
i	Consult instructions for use
(Do not re-use
Qty.	Number of units in the package
×	Keep away from sunlight
REF	Article number
SN	Serial number
LOT	Batch number
ONR	Order number
Rx only	Caution: Federal law restricts this device to sale by or on the order of a physician
\sim	Date of manufacture/sterilisation (YYYY-MM or YYYY-MM-DD)
\sum	Use by date (YYYY-MM-DD)
	Fragile, handle with care
Ť	Keep dry
	Do not use if package is damaged and consult instructions for use
STERILEEO	Sterilized using Ethylene Oxide
sterile r	Sterilized using irradiation
NON STERILE	Non-sterile
MD	Medical Device
UDI	UDI Number
MAT	Material Number
	Contains hazardous substances

Further information on the symbols glossary can be found on our website.

1. General

This document contains information about the implant system and the related instruments.

The package inserts provided with the implants and instruments do not contain all of the information necessary for selection and application. For proper handling, refer to other product–related instructions, such as the instructions on the surgical technique associated with the relevant system, as well as the special handling recommendations and product labels, where applicable. Refer to the identification tag on the implant and/or the packaging label for the definitive identification information, such as system compatibility, article number, materials and shelf life. Refer to the relevant surgical technique for information on allocating and handling the instruments to be used for the implantation. You should also take advantage of the training courses and printed materials provided for your information. To learn more, please contact the Waldemar Link GmbH & Co. KG sales office or your field representative.

2. Brief description

This cementless implant system by Waldemar Link GmbH & Co. KG is intended for the partial or complete replacement of a diseased joint or a diseased bone region. It consist of defined components that can be combined with each other in accordance with their approved uses.

The selection and application of the devices presuppose standard training for orthopedic and surgical specialists and suitable experience with orthopedic and surgical procedures.

3. Indications and contraindications

Indications:

The LINK MP Monoblock Hip Prosthesis is indicated for patients with Mobility-limiting diseases, fractures or defects of the hip joint or proximal femur, which cannot be treated by conservative or osteosynthetic procedures

The LINK MP Monoblock Hip Prosthesis is indicated for the following conditions:

- Revision of loosened femoral prosthesis components involving extensive bone resorption of the proximal femur and widening of the medullary cavity or marked thinning of proximal femoral cortical bone the fixation of a new standard prosthesis is not possible
- Revision of loosened femoral prosthesis components by peri-/ subprosthetic fracture

The device is intended for cementless use.

Contraindications:

- Insufficient / inadequate bone mass- or quality, which prevents a stable anchorage of the prosthesis.
- Poor general state of health
- Acute or chronic infections, local and systemic
- Allergies to (implant) materials
- Distinctive muscular, nerve, vascular or other diseases, which put the affected limb at risk.

4. Implant handling

The implants are supplied sterile (gamma sterilization, at least 25 kGy) as single-use devices in individual packages. The packaging may contain protective components for the implants. These components are not intended for implantation.

The Implants should always be stored in their unopened protective packaging. Examine the packaging for damage before using the implant. Damaged packaging can have an adverse effect on both the sterility as well as the proper performance of the implant, such that the implant must not be used. Check the use by date on the packaging label. Implants with expired use by dates are no longer permitted to be used for implantation.

Observe the pertinent standards for the aseptic handling of devices during and after removal of the implant from the packaging. When removing the packaging, make a record of the batch or serial numbers on the label, since this information is decisive for batch tracing. Selfadhesive labels with this information are enclosed with every package for your convenience.

Caution!

- Implants must be handled with great care and should not be modified or changed, even the smallest scratches and damages can considerably impair their stability or performance. Damaged implants are not permitted be used.
- Manipulations, such as vigorous bending, kinking or bending backward are not permitted to be performed on implants that have fastening elements (e.g. straps) for intraoperative adjustment.
- Surfaces provided for the connection of modular prosthetic components (cone, pins, screws) must not be damaged and may need to be cleaned with sterile liquid and dried before being joined together, so that neither blood nor any other coating impairs any of the connections, which could compromise the reliability of the connection.
- When the implants are used in combination with ceramic components, the ceramic components must be in a brand new, unused and undamaged condition.
- In the case of broken ceramic components, the ceramic particles must be removed completely, since otherwise increased abrasion can be expected.
- Packaging and implants to be discarded must be handled in compliance with national and local regulations for hospital disposal.

5. Storage

The implants must be stored in the undamaged original packaging in buildings with adequate protection against damage due to impacts, frost, humidity, excessive heat, and direct sunlight.

The sterile status of the instruments depends, inter alia, on the sterile items packaging and the prevailing storage conditions and must be established together with the operator's hygiene officer on a case-bycase basis. The instruments shall be stored in a dry condition. Direct sunlight must be avoided.

6. Implant materials, implant selection, permissible combinations

Titanium and titanium alloys, cobalt-chromium-molybdenum alloys, aluminum oxide ceramics as well as aluminum composite ceramics are used as basic materials.

- Titanium Aluminum Vanadium Alloy (Ti6-AI-4V) according to ISO 5832-3
- "BIOLOX forte" aluminum oxide (Al2O3) ceramics according to ISO 6474-1 and "BIOLOX delta" aluminum oxide composite ceramics according to ISO 6474-2 by CeramTec AG
- Cobalt-based alloy (CoCrMo) according to ISO 5832-4/ASTM F-75 and ISO 5832-12/ASTM F-1537

Further information on the material compositions is available from the manufacturer upon request.

Please refer to the relevant surgical technique associated with the system and the identification on the packaging for further information on implant selection, permissible combination options and implant materials.

The LINK MP Monoblock Hip Prosthesis can be used with the LINK BiMobile Dual Mobility System (K171273).

Combinations with implants from other manufacturers and/or combinations with LINK implants that deviate from the surgical technique specifications have not been tested and are not permitted.

Permissible materials for tribological pairings:

- CoCrMo alloy / UHMWPE
- AI203 / UHMWPE
- Al203 / composite ceramics
- Composite ceramics / UHMWPE
- "Ceramic on metal" and "metal on metal" pairings are prohibited within the area of the bearing surface.

Range of motion of the implants:

The range of motion rises as the head diameter of the prosthesis increases. Waldemar Link GmbH & Co. KG offers specialized components that reduce the risk of dislocation in patients with an increased tendency to luxation. These include, for example, XL 28-size prosthesis heads made of CoCrMo. There is less range of motion than with a standard combination because the neck of the head and the cup shoulder reduce the range of motion.

Cones:

The male and female cones of modular stem connections, such as a hip prosthesis stem with a hip prosthesis head, must match. Special attention must be paid to correct cone coupling. The cones have been optimally matched to each other and are not permitted to be combined with devices by other manufacturers.

7. Possible risks and side effects

Surgery-related risks and side effects:

- Blood loss, allogeneic / autologous blood transfusions
- Swelling / hematomas
- Thrombosis / embolism / heart attack
- Impaired wound healing
- Infections
- Muscle and nerve damage
- Damage to blood vessels
- Postoperative pain

- Complications associated with the anesthesia applied
- Postoperative calcification

Implant-related risks and side effects:

- Intraoperative fractures
- Periprosthetic infection
- Allergic reactions to implant components and abraded particles
- Implant fractures / ceramic material fractures
- Implant loosening or subsidence
- Implant malpositioning / misalignment
- Reduced range of motion
- Luxation of joint components
- Discrepancies in the lengths of the extremities
- Premature wear and tear reoperation
- Postoperative pain, e.g. thigh pain
- Protrusion / Arrosion

8. Circumstances that can interfere with the success of an operation

- Severe osteoporosis
- Severe deformities
- Local bone tumors
- Systemic diseases
- Metabolic disorders
- Case history of infections and falls
- Drug dependency or abuse, including excessive alcohol and nicotine consumption
- Obesity
- Mental disorders or neuromuscular diseases
- · Heavy physical activities associated with strong vibrations
- Hypersensitivities

9. MRI safety information for implants

The LINK MP Hip Prosthesis has not been evaluated for safety and compatibility in the MR environment. It has not been tested for heating, migration, or image artifact in the MR environment. The safety of the LINK MP Hip Prosthesis in the MR environment is unknown. Scanning a patient who has this device may result in patient injury.

10. Important information

If the implantation of a LINK implant system is considered the best solution for the patient and one of the circumstances described in section 8 is applicable to the patient, it is necessary to advise the patient with regard to the anticipated effects that these circumstances could have on the success of the operation. It is further recommended that the patient be informed about measures that he or she can take to reduce the effects of such complications. All information provided to the patient should be documented in writing by the operating surgeon.

The patients should be instructed in detail about the limitations of the implants, especially about the effects of excessive stress caused by body weight and physical activity, among other things. They should be encouraged to adjust their activities accordingly.

Proper selection, placement and fixation of the devices are decisive factors, which will determine the life of the implant.

Enquiries of any kind should be directed to Waldemar Link GmbH & Co. KG. The same applies for requests for further information on the devices.

11. Preoperative planning

Preoperative planning provides important information to identify the appropriate implant system and select the components of a system. Make sure that all components required for the operation are laid out and ready in the operating room. Test prostheses to verify proper fit (where applicable) and additional implants should be kept at the ready, in case other sizes are needed or the intended implant cannot be used. All LINK instruments necessary for the implantation must be on-hand and intact. If endoprosthetic joint replacement is indicated, then it must be taken into consideration, along with the overall picture of the patient:

- that all non-surgical and surgical treatment alternatives for the joint disease have been considered
- that artificial joint replacement performance is categorically inferior to natural joint performance, and an indication-related improvement in the preoperative condition is the only aim here
- that an artificial joint may loosen due to stress, wear and tear, and infection, or luxation or dislocation may occur
- that revision surgery, which under certain circumstances may exclude the possibility of restoring joint function, may be necessary due to loosening of the implant
- that if the selection of cementless implants is indicated, the biological age of the patient, among other things, must be considered
- that the patient consents to undergo the operation and accepts the risks involved
- that if load-transferring bone cement and/or bone structures are damaged, then the loosening of the components, bone and implant fractures, as well as other serious complications cannot be ruled out
- that if the patient is suspected of having allergies and tests positive on the applicable tests, then the patient's foreign body sensitivities (material tolerances) must be examined

Generally, the mechanical failure or fracture of joint replacement prosthesis is a rare exception. However, this cannot be excluded with absolute certainty. This may be due to stress on the implant and prosthesis as the result of a fall or accident, among other things. If the bone area where the implant is anchored is altered in such a way that the prosthesis is no longer able to withstand normal stress and an area of the prosthesis becomes subject to a stress imbalance, then a mechanical failure of the implant system may result. Such stress imbalances may also occur if the anchoring elements for the joint replacement implant are obliged to form a bridge over larger bone deficiencies without optimal reinforcement of the bone.

It is recommended that the implant with the largest possible anchoring elements is used.

Proper preparation for surgical procedures also includes the functional testing of implants and instruments prior to use.

12. Postoperative phase

In addition to movement and muscle training, special attention must be paid to carefully instructing the patient during the postoperative phase. Physician-supervised postoperative monitoring of healing progress is recommended. Where applicable, patients should also be advised on how to avoid overstraining themselves.

13. Reprocessing

The implants are supplied as sterile single-use devices. Implants that have already been implanted are not permitted to be reprocessed.

14. Implant resterilization

Our implants are designed for single-use only. Resterilization by the user is not permitted. Implants, as well as their materials are not suitable to be resterilized. Unpredictable degradations may occur in these implants during resterilization.

15. Instruments

Please refer to the description in the reprocessing instructions US-H50 for the:

- initial use
- performance test
- maintenance
- manual cleaning
- reprocessing
- sterilization
- servicing
- transport

16. Complaints about our products

Complaints of whatever kind must be filed with Waldemar Link at: complaint@linkhh.de.

When filing a complaint, always quote the name or REF number of the relevant component along with the LOT and SN number, your name and your contact address.

17. Servicing

Medical devices and instruments that are sent in for servicing must be processed beforehand in such a way that they cannot constitute a hazard to third parties. We will be pleased to provide you with further information about special instrument sets, their applications, disassembly, cleaning, and care.

Waldemar Link GmbH & Co. KG and/or other corporate affiliated entities own, use or have applied for the following trademarks in many jurisdictions: LINK, MP, BiMobile Other trademarks and trade names may be used in this document to refer to either the entities claiming the marks and/or names or their products and are the property of their respective owners.