



C.F.P. II Hip System Anatomically Adapted & Neck Preserving



Product Rationale



"The most important advancement in total hip arthroplasty in the last 50 years has been the adaption of the femoral components to the anatomy of the femur."¹



LINK is one of the pioneers of anatomical and neck preserving prosthesis designs and has decades of experience in the development of these stem types.

The C.F.P.* stem which was introduced in 1998, has since then strongly marked the further development in the world of short stems. Today we are looking back at an impressive track record with survival rates of up to 98,3 % after 11 years.²

In the C.F.P. II stem this legacy continues by taking clinically proven design features and combining them with today's requirements of a modern short stem.

The specific implant and instrument design conserves bone and soft tissue and adapts to the natural anatomy of the femur.³ Therefore the C.F.P. II equally lives up to its name and embodies our anatomical principles of bone preservation and "anatomy defines shape".

The system is accompanied by a compact, ergonomic instrument set, which enables the surgeon to perform the arthroplasty smoothly and efficiently, regardless of the favored surgical approach.

* Collum Femoris Preserving



Features of the C.F.P. II Hip System





Anatomical Shape



LINK HX (CaP) Coating



Bone and Soft Tissue Preserving³



MIS Support

LINK HX (CaP) Coating -Surface Technology for Long Term Fixation

The HX-Coating is an osteoconductive CaP coating, approximately 15 µm thick which is applied in an electrochemical process. Due to the ultra-thin and highly soluble HX overcoat, the porous cell structure of the underlying substrate is retained open. This promotes osteoconduction and allows a stable osseointegration for secondary stability.⁸



Features and Benefits





Anatomical Implant Design

The anatomical shape adapts to the natural femoral anatomy and effectively reduces stress risers.^{5, 6, 7}

Polished and flattened neck

for higher ROM and less wear.



Ribbed profile for excellent primary and secondary stability

The clinically proven ribbed structure serves to anchor the stem in the compressed cancellous bone and provides high primary and secondary stability.^{5, 10, 11}



The distal stem section facilitates implantation while also reducing the incidence of thigh pain, which can be caused by intramedullary fixation of a hip implant.^{12, 13} LINK HX (CaP) coating The osteoconductive surface promotes bone ongrowth.⁸

Substrat

Features and Benefits





Anatomical Compressor Design

Anatomically shaped stems require anatomically shaped instruments. The compressors of the C.F.P. II system strictly follow the anatomical stem design.

High fixation zones

Concave AP cross section for maximum bone preservation and engagement of the proximal ribs.

Press-fit

The compressors and implants are matched to provide optimal press-fit.



Terrace profile of the compressors

The C.F.P. II stem is anchored in a bed of compacted cancellous bone. The teeth of the compressors ensure that the cancellous substance is firmly compacted.^{4,9}

Rounded distal tip Safe guidance and to avoid via falsa.





Additional Information

Catalogs on request: E-mail customer@linkhh.de



For more information please register for our LINK Media Library (linkorthopaedics.com)



Cfp2opvideo.linkorthopaedics.com



Literature / References

- 1 W.T. Stillwell. (1987). The Art of Total Hip Arthroplasty. Grune & Stratton, pp. 296.
- 2 Kendoff D, Citak M, Egidy CC, O'Loughlin PF, Gehrke T. (2013). Eleven-year results of the anatomic coated CFP stem in primary total hip arthroplasty. Journal of Arthroplasty, 28(6), pp. 1047-51.
- 3 Vidalain, J. P., et al. (2011). The Corail Hip System. A practical approach based on 25 years of experience. Springer Heidelberg. pp. 54.
- 4 Pipino, F., Keller, A. (2006). Tissue-sparing surgery: 25 years' experience with femoral neck preserving hip arthroplasty. Journal of Orthopaedics and Traumatology, 7(1), pp. 36-41.
- 5 Langhans, M., Hofman, D., Ecke, H., & Nietert, M. (1992). Der Einfluß der Formgebung des Prothesenschaftes auf die Beanspruchung des proximalen Femurs. Unfallchirurgie, 18(5), pp. 266-273.
- 6 Noble, P., Alexander, J., Lindahl, L., Yew, D., Granberry, W., & Tullos, H. (1988). The anatomic basis of femoral component design. Clinical Orthopaedics and Related Research (235), pp. 148-165.
- 7 Denaro, V., & Fornasier, V. (2000). Fill, fit and conformation an anatomical and morphometric study of a hip component in total hip arthroplasty (Rippen-Link). European Journal of Orthopaedic Surgery & Traumatology, 10(4), pp. 239-247.
- 8 Palm, L., Jacobsson, S., & Ivarsson, I. (2002). Hydroxyapatite coating improves 8- to 10-year performance of the link RS cementless femoral stem. The Journal of Arthroplasty, 17(2), pp. 172-175.
- 9 DiGiovanni, C.W., Garvin, K.L., Pellicci, P.M. (1999). Femoral preparation in cemented total hip arthroplasty: reaming or broaching? Journal of the American Academy of Orthopaedic Surgeons, 7(6), pp.349-357.
- 10 Schill S, Thabe H. (2000). Long- and Mid-Term Results of the Cementless Link Prosthetic System in Combination with the Ribbed Stem and Screw-in Cup, Type "V". Orthopädische Praxis, 36, pp. 160-167.
- 11 Thabe H, Wolfram U, Schill S. (1993). Medium-term results using the cement-free link endoprosthesis. Ribbed shaft V socket. Zeitschrift fur Orthopädie und ihre Grenzgebiete, 131(6), pp. 568-573.
- 12 Petrou, G., Gavras, M., Diamantopoulos, M., Kapetsis, T., Kremmydas, N., & Kouzoupis, A. (1994). Uncemented total hip replacements and thigh pain. Archives of Orthopaedic and Trauma Surgery, 113(6), pp. 322-326.
- 13 Khanuja, H., Vakil, J., Goddard, M., & Mont, M. (2011). Cementless femoral fixation in total hip arthroplasty. The Journal of Bone & Joint Surgery, 93(5), pp. 500-509.

LINK ITALIA S.P.A., Milano

Products shown in this document may not be available in your country. The product availability is subject to the approval and/or registration regulations of the respective country. Please contact Waldemar Link GmbH & Co. KG (customer@linkhh.de) if you have questions about the availability of LINK products in your country.

Waldemar Link GmbH & Co. KG and/or other corporate affiliated entities own, use or have applied for the following trademarks in many jurisdictions: LINK, BiMobile, SP II, Modell Lubinus, E-Dur, EndoDur, T.O.P. II, BetaCup, CombiCup PF, CombiCup SC, CombiCup R, MobileLink, C.F.P., LCU, SP-CL, LCP, MIT-H, Endo-Model, Endo-Model SL, MP, MEGASYSTEM-C, GEMINI SL, SPAR-K, LCK, Link OptiStem, HX, TiCaP, X-LINKed, PorAg, LINK PorEx, BiPorEx, PorEx-Z, TrabecuLink, Tilastan, customLINK, RescueSleeve, Stactip, VACUCAST.

Other trademarks and trade names may be used in this document to refer to either the entities claiming the marks and/or names or their products and are the property of their respective owners.

© LINK 6712_CFP II-Product-EN-2021-07-002 MAR-02624 2.0

CE 0426

LINK ITALIA S.p.A.

via Cascina Belcasule, 11 - 20141 Milano - Italy Phone +39 02 535421 - Fax: +39 02 53542350 infolink@linkitaliaspa.it - www.linkorthopaedics.com

